



May 20, 2010

Dear Valued Customer,

Please use the following instructions to run liquid Influenza Proficiency Samples on the OSOM Influenza A & B Test Kit:

- 1) Add 135uL of flu proficiency sample to a test tube (provided in the test kit).
- 2) Add 135uL of extraction buffer to the test tube (provided in the test kit) and mix.
- 3) Add a Test Stick and incubate for 10 minutes.

NOTE: 135uL disposable pipettes are available from the Genzyme Diagnostics Technical Support Team, 800 332 1042 option #3.

If you have any questions or require additional assistance, please contact a member of our Technical Support Team at (800) 332-1042.